

UNIVERSAL BED HOLDER

IN POLYPROPYLENE

FOR THE ANTI-REFLUX URSEC® URINAL

Finally a multi-functional Urinal Holder with cover designed to be:

- A Urinal Holder hung or fixed at a variable height against the side of the bed
- A urine bag holder on the side of the bed
- A carrying box, a foldable displayer containing URSEC® with its documentation
- A stow away box easily stockable

EASY TO CLEAN AND STERILIZABLE AT 130°C (266°F)

1/ Carrying box and displayer



While being transported or during stockage, **URSEC** is maintained inside the carrying box thanks to the removable metallic fastener (1) inserted in the dedicated holes (2) on one of the two small flaps.

The hole in the medium flap (3) allows you to hang the displayer to present **URSEC** on the shelves of a shop.

For more information on the **URSEC** spillproof urinal ask for our documentation.


2/ Stow away box

Protected inside the box, **URSEC** can be stocked dust free.

The rectangular shape of the box allows for easy stacking and efficient stockage.

BED HOLDER INSTALLATION

To hang it on the side of a bed or a wheelchair: remove the hook like metallic fastener (1) from its shipping position (2) and insert it in the two holes (4) (separated by 70 mm) of the case, choosing wisely the ones most suitable with the height desired by the bedridden. Another possibility is to attach the case with the help of two screws already partially inserted in the wood of the bed using the two holes with buttonhole (5) (separated by 133 mm).

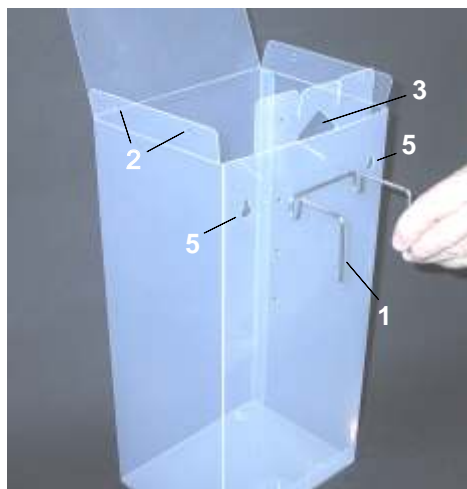

Made exclusively by **GOUGET®** (Paris - France) - www.gouget.com

- Patent pending -

Distributed by: